



## Microsoft Update and PHP Update Monthly Notification

**Summary:** This notification contains information about Microsoft security updates released this month and approved for implementation on the ISM nStream and VaultStream platform. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the nStream G3, nCareRX/MX, EasyLink Router, Thin Client, Status Board Engine, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v8.x and up VaultStream systems.

**Product:** nStreams, nCares, and Thin Clients connected to a network and VaultStream, as well as the Status Board Engine.

**Date:** February 15, 2018

**Critical Security Bulletins (Obtained from Microsoft):**

NOTE: Image Stream Medical **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

**Approved Updates:**

February 2018 Updates		
KB Number	Platform	Title
KB2952664	nStream	Update for Windows 7
KB890830	nStream	Windows Malicious Software Removal Tool - February 2018
KB890830	VaultStream	Windows Malicious Software Removal Tool x64 - February 2018
KB4019276	ERL v8.0	Update for WES09 and POSReady 2009
KB4034044	ELR v8.0	2018-02 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4019990	nCareRX/MX/GX-SBE	Update for Windows Embedded Standard 7 for x64-based Systems (KB4019990)
KB4074587	nCareRX/MX/GX-SBE	2018-02 Security Only Quality Update for Windows Embedded Standard 7 for x64-based Systems
KB4074598	nCareRX/MX/GX-SBE	2018-02 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x64-based Systems
KB 4074805	nCareRX/MX/GX-SBE	2018-02 Preview of Quality Rollup for .NET Framework 3.5.1 on Windows Server 2008 R2 for Itanium-based Systems
		Contains KB4054998
KB2972106	nCareRX/MX/GX-SBE	Security Update for Microsoft .NET Framework 4 on Windows Server 2003, Vista, Windows 7, Server 2008, Server 2008 R2 x64
		(only) msipatchregfix-amd64_5011cb29b096fb674a4795ee8fc2f7fdad33863a.exe
KB4074587	ThinClient-ELR v8.1	2018-02 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems
KB4074598	ELR v8.1	2018-02 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x86-based Systems (KB4074598)
<b>PHP Updates</b>		
None		

**For further questions concerning this notification please call Technical Support at (866) 376-6800.**