



Microsoft Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the ISM nStream and VaultStream platform. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the nStream G3, nCareRX/MX, EasyLink Router, Thin Client, Status Board Engine, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v8.x and up VaultStream systems.

Product: nStreams, nCares, and Thin Clients connected to a network and VaultStream, as well as the Status Board Engine.

Date: January 29, 2018

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Image Stream Medical **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

January 2018 Updates		
KB Number	Platform	Title
KB890830	nStream	Windows Malicious Software Removal Tool - January 2018
KB890830	VaultStream	Windows Malicious Software Removal Tool x64 - January 2018
KB890830	VaultStream	Windows Malicious Software Removal Tool for Windows 8, 8.1, 10 and Windows Server 2012, 2012 R2, 2016 x64 Edition - January 2018
KB915597	VaultStream	Definition Update for Windows Defender - (Definition 1.259.1345.0)
KB915597	VaultStream	Definition Update for Windows Defender - (Definition 1.259.1638.0)
KB4033342	VaultStream	Microsoft .NET Framework 4.7.1 for Windows 7 and Windows Server 2008 R2 for x64
KB4033369	VaultStream	Microsoft .NET Framework 4.7.1 for Windows 8.1 and Windows Server 2012 R2 for x64
KB4057270	nCareRX/MX/GX-SBE	2018-01 Preview of Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1 on Windows 7 and Server 2008 R2 for x64
KB4055532	nCareRX/MX/GX-SBE	2018-01 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1 on Windows 7 and Server 2008 R2 for x64
KB4055269	nCareRX/MX/GX-SBE	2018-01 Security Only Update for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1 on Windows 7 and Server 2008 R2 for x64
KB4056359	nCareRX/MX/GX-SBE	Update for Windows Embedded POSReady 7 for x64-based Systems
KB4073578	nCareRX/MX/GX-SBE	2018-01 Update for Windows Embedded Standard 7 for x64-based Systems
KB4056894	nCareRX/MX/GX-SBE	2018-01 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x64-based Systems
KB4056897	nCareRX/MX/GX-SBE	2018-01 Security Only Quality Update for Windows Embedded Standard 7 for x64-based Systems
KB4073578	ThinClient-ELR v8.1	2018-01 Update for Windows Embedded Standard 7 for x86-based Systems
KB4056894	ThinClient-ELR v8.1	2018-01 Security Monthly Quality Rollup for Windows Embedded Standard

		7 for x86-based Systems
KB4056897	ThinClient-ELR v8.1	2018-01 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems
PHP Updates		
5.6.33		

Not Approved Updates:

January 2018 Updates		
KB Number	Platform	Title
KB4054178	ELR v8.0	2018-01 Security Only Update for .NET Framework 2.0 on WES09 and POSReady 2009
KB4055229	ELR v8.0	2018-01 Security Only Update for .NET Framework 3.0 on WES09 and POSReady 2009
KB4054173	ELR v8.0	2018-01 Security Only Update for .NET Framework 4 on WES09 and POSReady 2009
KB4056568	ThinClient	Cumulative Security Update for Internet Explorer 9 for Windows Server 2008

For further questions concerning this notification please call Technical Support at (866) 376-6800.