



Microsoft Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the ISM nStream and VaultStream platform. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the nStream G3, nCareRX/MX, EasyLink Router, Thin Client, Status Board Engine, MedPresence Stand-Alone, All-In-One Client, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v8.x and up VaultStream systems.

Product: nStreams, nCares, and Thin Clients connected to a network and VaultStream, as well as the Status Board Engine.

Date: May 15, 2018

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Image Stream Medical **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

May 2018 Updates		
KB Number	Platform	Title
nStream v5.4 Updates		
KB4099633	nStreamG3	2018-05 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1 on Windows 7
KB4103718	nStreamG3	2018-05 Security Monthly Quality Rollup for Windows 7 for x86-based Systems
KB890830	nStreamG3	Windows Malicious Software Removal Tool - May 2018
VaultStream v5.3, 8.0.x, v9.0.x Updates		
KB4099633	VaultStream	2018-05 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1 on Windows 7 and Server 2008 R2 for x64
KB4103718	VaultStream	2018-05 Security Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems
KB4103768	VaultStream	Cumulative Security Update for Internet Explorer 11 for Windows Server 2008 R2 for x64-based Systems
KB890830	VaultStream	Windows Malicious Software Removal Tool x64 - May 2018
EasyLink Router v8.0 Updates		
KB4089453	ELR v8.0	2018-05 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4056564	ELR v8.0	2018-05 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4131188	ELR v8.0	2018-05 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4101477	ELR v8.0	2018-05 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4134651	ELR v8.0	2018-05 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4130957	ELR v8.0	2018-05 Security Update for WES09 and POSReady 2009 for x86-based Systems

nCareRX v8.0 and v8.1, nCareMX/nStreamGX v8.0, v9.0, and Status Board Engine, 8.0 and v8.1 Updates		
KB4099637	nCare-SBE	2018-05 Security Only Update for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1 on Windows 7 and Server 2008 R2 for x64
	nCare-SBE	windows6.1-kb4019990-x64_35cc310e81ef23439ba0ec1f11d7b71dd34adfe5.msu
	nCare-SBE	windows6.1-kb4095514-x64_34fc4390d8839c2390c7dd6fee369086f516cf6d.msu
KB4099633	nCare-SBE	2018-05 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1 on Windows 7 and Server 2008 R2 for x64
	nCare-SBE	windows6.1-kb4095874-x64_13904512d1614ba1a336061e90f7078becf3be5a.msu
KB4103718	nCare-SBE	2018-05 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x64-based Systems
	nCare-SBE	windows6.1-kb4103718-x64_c051268978faef39e21863a95ea2452ecbc0936d.msu
	nCare-SBE	pclearstalecache_264c3460a3ee95d831aa512b80bc8cc6aaa2d218.exe
KB4103712	nCare-SBE	2018-05 Security Only Quality Update for Windows Embedded Standard 7 for x64-based Systems
HP Thin Client v8.0 and EasyLink Router v8.1 Updates		
KB4103718	ThinClient-ELR v8.1	2018-05 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x86-based Systems
	ThinClient-ELR v8.1	pclearstalecache_30ad965244b47cf70e513cc3fe228a5936629ab2.exe
	ThinClient-ELR v8.1	windows6.1-kb4103718-x86_1e4b61004905b531e1541f829799c2ca88c15a01.msu
KB4103712	ThinClient-ELR v8.1	2018-05 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems
MedPresence Stand-Alone v9.0.x and All-In-One Client (works with v9.0.x and 10.x)		
KB4103723	MedPresence-All-In-One	2018-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4103723	MedPresence-All-In-One	2018-05 Delta Update for Windows 10 Version 1607 for x64-based Systems
KB4103729	MedPresence-All-In-One	2018-05 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
ES4K v10.0.x - BlizCon		
KB4103723	BlizCon	2018-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4132216	BlizCon	2018-05 Update for Windows 10 Version 1607 for x64-based Systems
KB4103729	BlizCon	2018-05 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB890830	BlizCon	Windows Malicious Software Removal Tool x64 - May 2018
KB4033631	BlizCon	Update for Windows 10 Version 1607 for x64-based Systems
KB4049411	BlizCon	Update for Windows 10 Version 1607 for x64-based Systems
ES4K v10.0.x - nCareRX2		
KB4103720	nCareRX2	2018-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB890830	nCareRX2	Windows Malicious Software Removal Tool x64 - May 2018
KB4132216	nCareRX2	2018-05 Update for Windows 10 Version 1607 for x64-based Systems
KB4103723	nCareRX2	2018-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4103729	nCareRX2	2018-05 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
ES4K v10.0.x - LSSB2		
KB4132216	LSSB2	2018-05 Update for Windows 10 Version 1607 for x64-based Systems
KB4103720	LSSB2	2018-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems

KB4103723	LSSB2	2018-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4103729	LSSB2	2018-05 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB890830	LSSB2	Windows Malicious Software Removal Tool x64 - May 2018
KB4093120	LSSB2	2018-04 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB890830	LSSB2	Windows Malicious Software Removal Tool x64 - April 2018
KB4093137	LSSB2	2018-04 Update for Windows 10 Version 1607 for x64-based Systems
KB4093110	LSSB2	2018-04 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4093119	LSSB2	2018-04 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4033631	LSSB2	Update for Windows 10 Version 1607 for x64-based Systems
KB4049411	LSSB2	Update for Windows 10 Version 1607 for x64-based Systems
PHP Updates		
None		

For further questions concerning this notification please call Technical Support at (866) 376-6800.