



Microsoft Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Gyrus ACMI, Inc., ES-IP, ES4K, LiveStream™, nCare™ and VaultStream™ platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite™ HD, ES4K, ES-IP iOR systems, nCare MX/RX recorders, nCare4K, EasyLink™ Router, LiveStream StatusBoard Engine, and VaultStream systems and its KE Modules.

Also included is information on the latest PHP updates approved for the v10.x and up VaultStream systems.

Products: ES-IP and ES4K iOR systems and nCare recorders, connected to a network and VaultStream server software, and the LiveStream StatusBoard Engine.

Date: September 2025

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Gyrus ACMI, Inc. **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

MedPresense Carts are no longer supported due to EOL

Approved Updates:

September 2025 Updates		
Platform	KB	Title
VaultStream v10.x Updates – Windows Server 2016		
VaultStream 2016	KB5065687	2025-09 Servicing Stack Update for Windows Server 2016 for x64-based Systems (KB5065687)
VaultStream v10.x Updates – Windows Server 2019		
VaultStream 2019	KB5065955	2025-09 Cumulative Update for .NET Framework 3.5, 4.7.2 and 4.8 for Windows Server 2019 for x64 (KB5065955)
VaultStream 2019	KB5065428	2025-09 Cumulative Update for Windows Server 2019 (1809) for x64-based Systems (KB5065428)
VaultStream v11.x Updates – Windows Server 2022		
VaultStream 2019	KB5065955	2025-09 Cumulative Update for .NET Framework 3.5, 4.7.2 and 4.8 for Windows Server 2019 for x64 (KB5065955)
VaultStream 2019	KB5065428	2025-09 Cumulative Update for Windows Server 2019 (1809) for x64-based Systems (KB5065428)
nCare MX v10.2.3		
nCare MX	KB5065427	2025-09 Cumulative Update for Windows 10 Version 1607 for x64-based Systems (KB5065427)
nCare MX	KB5065687	2025-09 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems (KB5065687)
nCare MX v10.4		
nCare MX	KB5065427	2025-09 Cumulative Update for Windows 10 Version 1607 for x64-based Systems (KB5065427)
nCare MX	KB5065687	2025-09 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems (KB5065687)
nCare 4K v11.3		
nCare4K	KB5065956	2025-09 Cumulative Update for .NET Framework 3.5, 4.8 and 4.8.1 for Windows 10 Version 21H2 for x64 (KB5065956)
nCare4K	KB5065429	2025-09 Cumulative Update for Windows 10 Version 21H2 for x64-based Systems (KB5065429)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.674.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.664.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.647.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.633.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.630.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.626.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.621.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.615.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.600.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.592.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.575.0) - Current Channel (Broad)

nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.573.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.565.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.559.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.552.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.546.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.539.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.528.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.528.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.524.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.515.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.509.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.508.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.501.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.492.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.490.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.478.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.466.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.454.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.448.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.437.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.428.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.426.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.417.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.408.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.401.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.393.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602

		(Version 1.435.383.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.370.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.369.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.365.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.359.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.357.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.354.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.348.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.345.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.332.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.324.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.307.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.302.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.300.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.286.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.281.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.278.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.261.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.250.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.249.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.247.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.242.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.236.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.235.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.230.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.225.0) - Current Channel (Broad)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.219.0) - Current Channel (Broad)

nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.207.0) - Current Channel (Broad)	
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.205.0) - Current Channel (Broad)	
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.197.0) - Current Channel (Broad)	
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.186.0) - Current Channel (Broad)	
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.183.0) - Current Channel (Broad)	
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.171.0) - Current Channel (Broad)	
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.165.0) - Current Channel (Broad)	
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.153.0) - Current Channel (Broad)	
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.146.0) - Current Channel (Broad)	
ES4K v10.4 ES4K Controller (a.k.a. BlizCon)			Security Intell
ES4K Controller	KB5065427	2025-09 Cumulative Update for Windows 10 Version 1607 for x64-based Systems (KB5065427)	
ES4K Controller	KB5065687	2025-09 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems (KB5065687)	
ES4K v10.4 ES4K Video Manager (a.k.a. nCare RX2)			
ES4K Video Manager	KB5065427	2025-09 Cumulative Update for Windows 10 Version 1607 for x64-based Systems (KB5065427)	
ES4K Video Manager	KB5065687	2025-09 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems (KB5065687)	
ES4K v10.3 LiveStream StatusBoard 2			
LiveStream StatusBoard 2	KB5065687	2025-09 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems (KB5065687)	
LiveStream StatusBoard 2	KB5065427	2025-09 Cumulative Update for Windows 10 Version 1607 for x64-based Systems (KB5065427)	
LiveStream StatusBoard 2	KB5065749	2025-09 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64 (KB5065749)	
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.667.0) - Current Channel (Broad)	
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.647.0) - Current Channel (Broad)	
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.633.0) - Current Channel (Broad)	
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.630.0) - Current Channel (Broad)	
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.628.0) - Current Channel (Broad)	
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.621.0) - Current Channel (Broad)	
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.620.0) - Current Channel (Broad)	
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.600.0) - Current Channel (Broad)	

LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.596.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.575.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.559.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.556.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.539.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.533.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.513.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.498.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.492.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.466.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.443.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.423.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.418.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.401.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.396.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.381.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.378.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.365.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.363.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.354.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.350.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.332.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.326.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.307.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.303.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.284.0) - Current Channel (Broad)

LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.281.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.261.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.244.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.230.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.221.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.198.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.183.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.178.0) - Current Channel (Broad)
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.153.0) - Current Channel (Broad)
ES-IP iOR system v11.4		
ES-IP iOR system	KB5066233	2025-09 .NET 8.0.20 Update for x64 Client (KB5066233)
ES-IP iOR system	KB4052623	Update for Microsoft Defender Antivirus antimalware platform - KB4052623 (Version 4.18.25080.5) - Current Channel (Staged)
MAJ-2533		
MAJ-2533	KB5065955	2025-09 Cumulative Update for .NET Framework 3.5, 4.7.2 and 4.8 for Windows 10 Version 1809 for x64 (KB5065955)
MAJ-2533	KB5065428	2025-09 Cumulative Update for Windows 10 Version 1809 for x64-based Systems (KB5065428)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.677.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.664.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.658.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.633.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.626.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.624.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.622.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.611.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.607.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.590.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.575.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.573.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.567.0) - Current Channel (Broad)

MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.559.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.552.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.546.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.536.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.528.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.524.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.513.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.509.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.508.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.496.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.492.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.490.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.485.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.466.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.461.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.437.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.436.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.417.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.413.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.393.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.383.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.370.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.369.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.367.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.359.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.357.0) - Current Channel (Broad)

MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.355.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.348.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.342.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.333.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.324.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.319.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.307.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.300.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.294.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.284.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.278.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.268.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.261.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.250.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.244.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.241.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.238.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.230.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.229.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.219.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.216.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.214.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.193.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.165.0) - Current Channel (Broad)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.435.146.0) - Current Channel (Broad)
CV Connectivity with KE v3.1.1 (VaultStream with Interfaces License) - Windows 10 (22H2) 64bit		
KE v3.1	KB5063709	2025-08 Cumulative Update for Windows 10 Version 22H2 for x64-based Systems (KB5063709)

KE v3.1	KB890830	Windows Malicious Software Removal Tool x64 - v5.135 (KB890830)
CV Connectivity with KE v3.1.1 (VaultStream with Interfaces License) – Windows Server 2016		
KE v3.1	KB5063871	2025-08 Cumulative Update for Windows Server 2016 for x64-based Systems (KB5063871)
KE v3.1	KB890830	Windows Malicious Software Removal Tool x64 - v5.135 (KB890830)
CV Connectivity with KE v3.4.0 (VaultStream with Interfaces License) - Windows Server 2019		
KE v3.4.0	KB5063877	2025-08 Cumulative Update for Windows Server 2019 (1809) for x64-based Systems (KB5063877)
KE v3.4.0	KB890830	Windows Malicious Software Removal Tool x64 - v5.135 (KB890830)
CV Connectivity with KE v3.4.0 (VaultStream with Interfaces License) - Windows Server 2022		
KE v3.4.0	KB5063880	2025-08 Cumulative Update for Microsoft server operating system version 21H2 for x64-based Systems (KB5063880)
KE v3.4.0	KB890830	Windows Malicious Software Removal Tool x64 - v5.135 (KB890830)
CV Connectivity with KE v3.4.0 (VaultStream with Interfaces License) - Windows Server 2025		
KE v3.4.0	KB5063709	2025-08 Cumulative Update for Windows 10 Version 22H2 for x64-based Systems (KB5063709)
KE v3.4.0	KB890830	Windows Malicious Software Removal Tool x64 - v5.135 (KB890830)
V10.2.2 was Tested with PHP version 7.3.33 V10.4 was Tested with PHP version 7.4.30 PHP 7.3.33 is available upon request for v10.2.2 VaultStreams only. PHP 7.4.33 is available upon request for v10.4 VaultStreams only. PHP V8.1.19 is available upon request v10.7 VaultStreams only. PHP V8.3.8 is available upon request v10.7 VaultStreams only. PHP V8.3.9 is available upon request v10.7 VaultStreams only		

For further questions concerning this notice, please call Technical Support at 1-800-848-9024